DermaFreezer®

EN - INSTRUCTIONS FOR USE
FR - MODE D'EMPLOI
DE - GEBRAUCHSANWEISUNG
NL - GEBRUIKSAANWIJZING
ES - MODO DE EMPLEO
PT - INSTRUÇÕES DE USO

IT - ISTRUZIONI PER L'USO
PL - INSTRUKCJA UŻYTKOWANIA
NO - BRUKSANVISNING
FI - KÄYTTÖOHJEET
SE - BRUKSANVISNING
CS - NÁVOD K POUŽITÍ

Instructions for use: Cryo professional

Portable cryosurgical system

For the removal of warts and other benign skin lesions

CAUTION

Read instruction for use carefully prior to using this product.
Use of the product in accordance with its instructions at all time.

Using the product in a way other than instructed, may lead to thermal reactions, skin damage, nerve damage and scar formation.

For external use only.

For use by medical professionals only.

COMPLAINTS AND CUSTOMERS SERVICE SUPPORT

Any serious adverse event related to this product must be declared to Koninklijke Utermöhlen NV and national health authority in your country. Contact Koninklijke Utermöhlen NV with any question or to report problems.

CONTENTS

- 1x Aerosol dispenser: Filled with 170 ml cryogen comprised of a mixture of dimethyl-ether, propane, butane and iso-butane. The cryogen does not damage the ozone layer.
- 1x Box containing foam applicators: The package includes 2 mm small foam applicators and/or 5 mm medium foam applicators. Applicators are for single use per patient only.
- 1x Instruction for use.

INTENDED USE / INDICATIONS

The product is used for the removal of various epidermal skin lesions. Different sizes of foam applicators supplied are to be used depending on the size of the lesion. Different types of skin lesions are treated with different freezing times.

WARNING

Do not use other brands of foam applicators. Use of other brands of foam applicators may affect the performance of the product and could lead to unwanted damage to the skin and underlying tissues.

The product is intended for use on patients older than 4 years of age with skin lesions. The product is indicated for treatment of the following skin lesions:

- Genital warts
- Molluscum Contagiosum
- Acrochordon (Skin tags)
- Verruca (Plantaris, Vulgaris, Plana)
- Seborrheic Keratosis
- Actinic Keratosis (facial and non-facial)
- Lentigo (facial and non-facial)

CONTRA INDICATIONS

Do not use cryo professional:

- When there is or has been a confirmed diagnosis of cancer and treatment for cancer.
- With diabetics and patients with circulatory problems.
- With patients with cryoglobulinaemia.
- With patients using immunosuppressant medicines.

WARNINGS & PRECAUTIONS

- Do not use in the vicinity of other medical or liquid gases.
- Do not use in the vicinity of or with diathermy equipment.
- Hypopigmentation and/or inflammatory hyperpigmentation may occur in highly pigmented and

- lightly pigmented skin.
- Deep freezing in regions of peripheral arteries such as in the extremities may result in necrosis distal to the frozen lesions.
 - When the dermis is not damaged by the cyrosurgery, scar formation is rare.
 - Do not saturate the foam applicator again during use. This will diminish the reservoir function of the foam applicator.
 - Do not activate the aerosol while the foam applicator is in contact with the patient.
 - Extremely flammable aerosol.
 - Pressurized container: May burst if heated.
 - Keep out of reach of children. Do not inhale spray.
 - Keep away from heat, hot surfaces, sparks, open flames and other ignition sources.
 - No smoking.
 - Do not pierce or burn, even after use.
 - Do not spray onto an open flame or other ignition source.
 - Protect from sunlight.
 - Do not expose to temperatures exceeding 50 °C/122 °F.
 - Dispose of contents/container in accordance with local regulations.
 - Care should be taken when using this product on pregnant or breast feeding women.

INDICATED FREEZING TIMES

Skin Lesion Type	Recommend Freezing Times
Ano-genital warts (Condyloma acuminatum)	40 seconds
Molluscum Contagiosum	20 seconds
Acrochordon (Skin tags)	40 seconds
Verruca plantaris	40 seconds
Verruca vulgaris	40 seconds
Verruca plana	20 seconds
Seborrheic keratosis	40 seconds
Actinic keratosis (facial)	15 seconds
Actinic keratosis (non-facial)	40 seconds
Lentigo (facial)	15 seconds
Lentigo (non-facial)	40 seconds

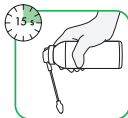
4


5

6 7

8

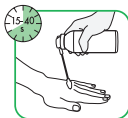8

INSTRUCTIONS FOR USE (STEP-BY-STEP)

- 1 Determine the type of skin lesion you are going to treat. Treat larger lesions (> 5 mm) in sections.
- 2 Apply the appropriate antiseptic treatment in order to avoid the risk of infection.
- 3 Determine the best way to treat the skin lesion. In case of warts, a keratolytic agent may need to be applied prior to the cryotherapy treatment; this will not affect the product performance.
- 4 Determine which foam applicator is to be used (2 mm or 5 mm) and insert it into the dispenser.
- 5 Remove the protective cap from the aerosol and activate the dispenser with the thumb or forefinger to saturate the foam applicator with cryogen until the droplets fall from the foam applicator. Hold the aerosol upright at all times while saturating the foam applicator. Keep the aerosol away from persons in the room.
- 6 Hold the aerosol to a 90° angle so that the foam applicator is pointing vertically downwards.
- 7 Wait for 15 seconds for the foam applicator to achieve its effective temperature.
- 8 Place the foam applicator on the skin lesion to be treated exerting only slight pressure. Apply the foam applicator to the lesion for the recommended treatment time as given in the schedule. During treatment hold the foam applicator vertically downwards (90° angle) at all times.
- 9 If necessary, you may repeat the treatment at up to 4 times with intervals of 2 weeks.

POTENTIAL ADVERSE REACTIONS

Ensure that the patient is informed about potential side effects.

The main side effects include:

- A painful burning or stinging sensation.
- Blistering.
- Hypopigmentation.
- Inflammatory hyperpigmentation.
- Skin atrophy.

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Caballero Martinez F, et.al. Cutaneous cryosurgery in family medicine: dimethyl-ether-propane spray vs liquid nitrogen. *Aten Primaria*. (1996); 18(5): 211-216

Graells J, Espinola A, Barrio C, Muñoz MD, Román A, Parellada N. Minor cutaneous ambulatory surgery and cryotherapy. Comparative study between a dermatologist and family physicians. *Actas Dermosifiliogr*. 2007;98(3):171-7

Thai KE, et.al. Cryosurgery for skin lesions. *Australia J Dermatol* (1999); 40(4): 175-184

Thai KE, Fergin P, Freeman M, Vinciullo C, Francis D, Spelman L, Murrell D, Anderson C, Weightman W, Reid C, Watson A, Foley P. A prospective study of the use of cryosurgery for the treatment of actinic keratoses. *Int J Dermatol*. 2004; 43(9):687-692

Wetmore SJ. Cryosurgery for common skin lesions: Treatment in family physician's offices. *Can Fam Physician*. (1999); 45: 964-974

Servoprax GmbH.
Am Marienbusch 9, 46485 Wesel
Germany, www.servoprax.de

Koninklijke Utermöhlen NV
De Overweg 1, 8471 ZA Wolvega
The Netherlands +31 (0)561 69 33 66

CE
0344

xxxxx - V1.20231003

DermaFreezer®