



# COMED®

## INFORMATION AND ASSEMBLY GUIDE

Thank you for choosing the COMED weight lifting device.  
Below you will find some tips for use as well as technical data.

### This kit contains :

- A bolt of  $\varnothing$  8 mm, length 80 mm
- A heavyweight of 2Kg or 5Kg
- An assembly guide

### Technical data :

Reference :	45 419 95 (2 Kg)	45 419 97 (5 Kg)
Weight dimensions (mm) : $\varnothing$ x Ep	180 x 18	240 x 25
Bolt dimensions (mm) : $\varnothing$ x L	8 x 80	8 x 80
Weight material :	Cast iron	Cast iron
Coating material :	Black Epoxy	Black Epoxy

### Assembly guide for the infusion foot :

1. Remove the original tube bolt.
2. Insert the tube of the infusion foot into the casters base.
3. Install the weight adjuster below the base.
4. Place the washer just below the weights and screw in the supplied bolt all the way. (You will need a wrench, flat or pipe,  $\varnothing$  13 mm)

### Magnifying lamp assembly guide :

Same instructions as for the infusion foot.

Feel free to use a rubber mallet to drive the tube into the base.

### Maintenance instructions :

Clean with soapy water or detergent.

For disinfection, we recommend that you disassemble the weight device and disinfect it with a disinfectant that does not attack epoxy. If the weighting device is damaged, dry it well after cleaning or disinfection (risk of rust).

