


# RVS-100

## Vital Signs Monitor + Thermometer

## Quick Start Guide



### How to connect the Bluetooth Thermometer at the RVS-100

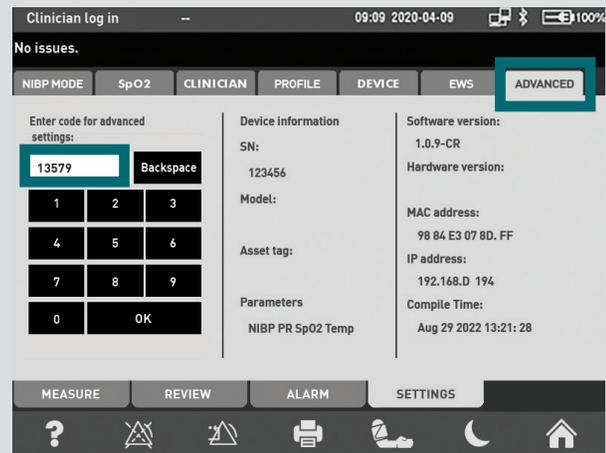

**STEP 1.** After pressing the power switch, the monitor will begin an automatic self-diagnostic test and start-up. During this process, the monitor will check the battery status and for error messages etc. The device will make a sound and the Riester logo will appear on the screen.

**STEP 2.** After the Riester logo disappears, the monitor will display the main user interface. Press **SETTINGS** and press **ADVANCED** tab.



**STEP 3.** Enter code 13579 and press **OK**.

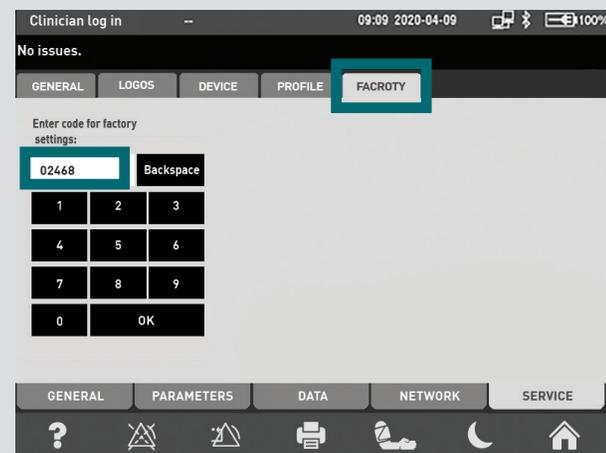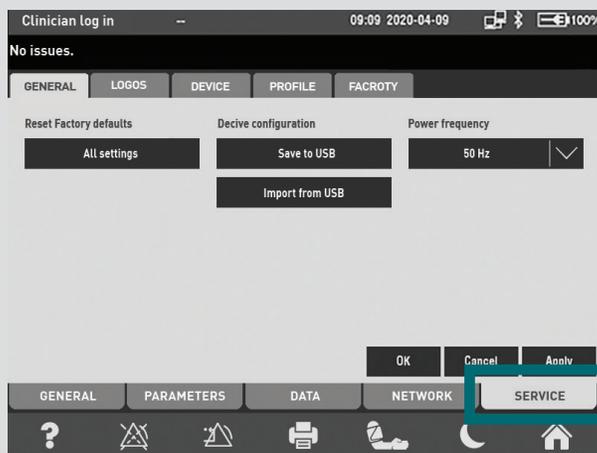

**STEP 4.** You are now in the **GENERAL** tab. Press **SERVICE** (Bottom Left).



**STEP 5.** Press the **FACTORY** tab.

**STEP 6.** Enter code 02468 and press **OK**.

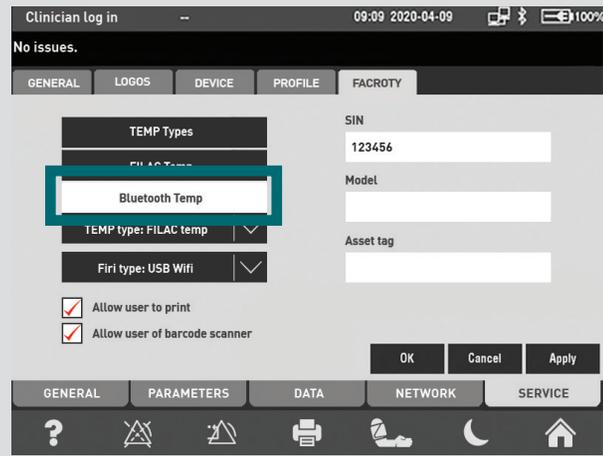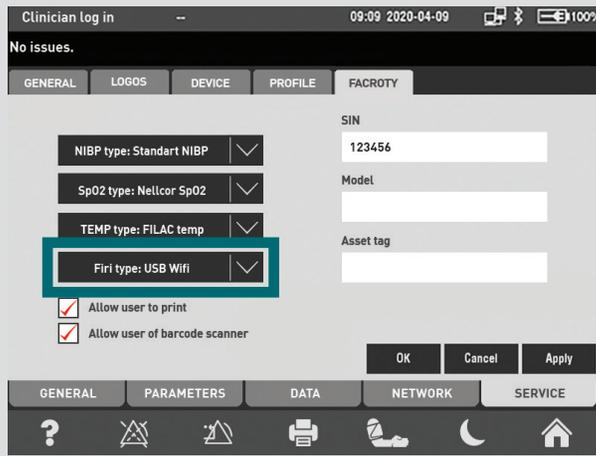

**STEP 7.** Press the TEMP type : FILAC Temp and choose Bluetooth temp on the options.



**STEP 8.** When the warning message, pop up appears, press OK and then press Apply. A pop up note will appear. Press OK button again.

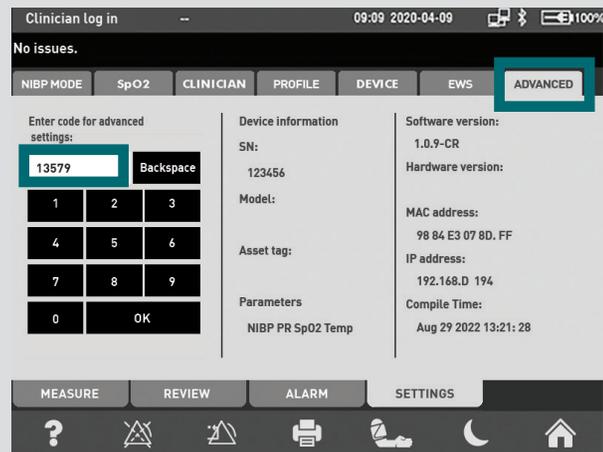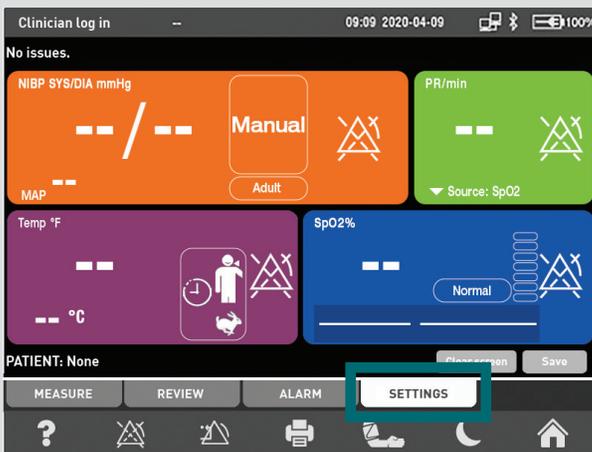

**STEP 9.** Turn monitor off and restart the RVS-100. A warning note will appear, press OK.

**STEP 10.** Press SETTINGS and press ADVANCED tab.

**STEP 11.** Enter code 13579 and press OK.

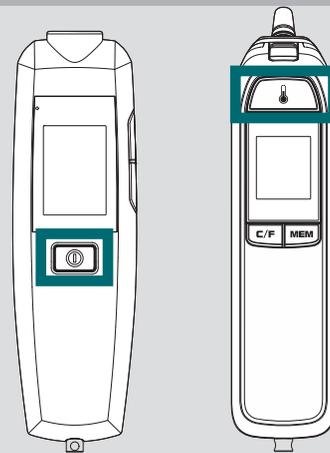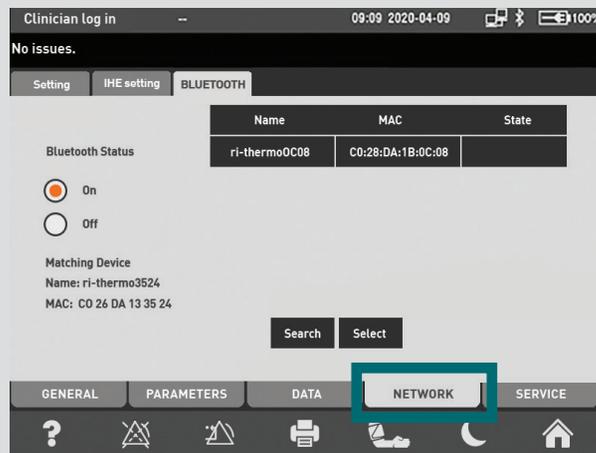

**STEP 12.** You are now in the NETWORK. Press the Bluetooth Status On.

**STEP 13.** Turn the Thermometer on and make a measurement.

**STEP 14.** When the MAC address of the thermometer (it is written on the backside on the label) should be appear, press Select.

99584 Rev. A 2023-09-06

Please visit [riester.de](http://riester.de) for more information



Refer to instructions for use for full functional usage



**Rudolf Riester GmbH** P.O. Box 35 | Bruckstraße 31 | 72417 Jungingen | Germany  
Tel.: (+49) 7477-9270-0 | Fax.: (+49) 7477-9270-70 | info@riester.de | www.riester.de

